



Ø3.2  
mm

## An Extended Solution for Narrow Ridges



# NICE Implant Drill Protocol

## Important:

- The length of the drill is measured from the tip to the middle of the depth marking



- Professional considerations may be required for adaptations of the drill protocol in specific cases
- In cases of extremely narrow ridges, additional surgical procedures may be required

	ø 2.0	ø 2.8
CODE	DRX2.0	DRX2.8
REF. NO	4204	4284

Scan to view  
Drill Protocol  
Movie



● Throughout entire implant length

● 3mm shorter than implant length

## Note

In cases of extremely hard bone, it is recommended to drill with ø 3.0 mm drill only through the cortical layer

<b>new!</b> Drill ø 3.0 mm	
CODE	DRX3.0
REF. NO	4306

